

GOBACK CATHETER VIDEO TRANSCRIPT

Tips and tricks: Crossing a tight iliac

When crossing a tight iliac arch with the 4 French device, it is recommended to use the following maneuver.

Start the procedure with a long 6 French sheath and the 120 centimetres 4 French GoBack catheter.

- Advance the GoBack until it cannot be advanced further over the iliac arch.
- Pull the sheath, together with the GoBack catheter, backwards a few centimetres.
- While holding the sheath in place, advance the GoBack catheter forward until it can no longer advance.
- Then push both the sheath and the GoBack together a few centimetres, and the GoBack will crossover smoothly.