

GOBACK CATHETER VIDEO TRANSCRIPT
Tips and Tricks - Which catheter to use

Here is a guide to selecting the optimal GoBack Catheter for your case.

- The 4 French catheter is stiffer and has greater pushability and torquability. We therefore recommend the use of the 4 French device wherever possible. This applies for all above-the-knee cases, so long as the vessel diameter is greater than 2.5 millimeter. You can also use the 4 French device when working below-the-knee on the peroneal and posterior tibial arteries. However, the 4 French GoBack is not ideal for the anterior tibial artery, as the clinician needs to navigate an angle which is too acute.
- The 2.9 French device is better suited for below the knee vessels. It is more flexible, and is effective in vessels with diameters as small as 2 millimeters. The trade-off, compared to the 4 French device, is a reduction in pushability.

This chart summarizes the GoBack Catheters' specifications.